Your Companion
Life Science Filtration

Certification of the i3 Membrane
quality management system according to DIN EN ISO 13485:2016

i3 Membrane GmbH - Bautzner Landstr. 45 - DE-01454 Radeberg
info@i3membrane.de - www.i3membrane.de
Phone: +49 (0)40-2576748-0 - Fax: +49 (0)40-2576748-48

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Information: **Duration of use:** The duration of use may be shorter if water quality is poor (rust, sand etc.).
Tankless systems: Do not install the filter/accessories on tankless (low pressure) systems
Disinfection by wiping: The filter can be disinfected by wiping with the permitted disinfectant (see www.i3membrane.de/desinfection)
Disposal: The filter is discarded with household trash

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Technical data: **Bacterial retention:** > 7 log per cm² (Brev. dim.)
Flow rate: 20 l/min at 5 bar (initial)
Duration of use: 50 days
Storage life: 5 years
Operating pressure: max 5 bar
Maximum operating temperature: 60 °C (max. 70 °C for 30 min.)
Measurements: 187 x Ø 40 mm (see www.i3membrane.de/accessories)
Accessories: (Adapter G1/2")
Item no. 2060050

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EN Instruction for use
Inline sterile filter for secure germ retention

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Artikelnummer

EN i3 FIVE inline sterile filter for secure germ retention

Installing

1. Disinfect your hands
2. Perform a surface disinfection of the quick release coupling
3. Open the packaging of the filter on the filter outlet side. Do NOT remove the filter completely from the packaging
4. Hold the filter in the opened packaging. Close the OUTLET SIDE of the filter to the associated quick release coupling and remove the packaging
5. Install the inlet side of the filter on the second quick release coupling
6. Check whether the filter is firmly installed in the quick release couplings
7. Open the water supply
8. Check the installation for leaks

Uninstalling

1. Close the water supply
2. Disconnect the filter on the inlet side
3. Then disconnect the filter on the outlet side
4. Dispose of the filter via the household trash

Do not use if the filter or the packaging is damaged

Do not use after the expiry date

Read the instructions thoroughly before use

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