Your Companion  
Life Science Filtration

Certification of the i3 Membrane quality management system according to DIN EN ISO 13485: 2016 and the products are EU 2017/745 (MDR) conform

i3 Membrane GmbH - Bautzner Landstr. 45 - DE-01454 Radeberg  
info@i3membrane.de - www.i3membrane.de  
Phone: +49 (0)40-2576748-0 - Fax: +49 (0)40-2576748-48

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**Information:**  
**Duration of use:** The duration of use may be shorter if water quality is poor (rust, sand etc.)  
**Tankless systems:** Do not install the filter /accessories on tankless (low pressure) systems  
**Disinfection by wiping:** The filter can be disinfected by wiping with the permitted disinfectant (see www.i3membrane.de/desinfection)  
**Disposal:** The filter is discarded with household trash

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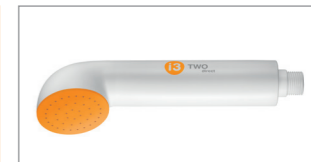
**EN** Instruction for use  
Sterile filter for use as a shower head to prevent infection

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EN **i3 TWO<sup>direct</sup> sterile filter for use as a shower head**

**Installing**

1. Unscrew the existing shower head from the shower hose if necessary
2. Disinfect your hands
3. Place a flat gasket in the shower hose
3. Open the transparent vacuum packaging of the filter on the filter inflow side
4. Screw the filter onto the shower hose by hand
5. Check the installed filter for leaks

**Uninstalling**

1. Unscrew the filter from the shower hose
2. Dispose of the filter with household trash

Do not use if the filter or the packaging is damaged

Do not use after the expiry date

Class I Medical device

Article number

Read the instructions thoroughly before use

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