REF 203300

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i3 TWO direct - Sterile filter for use as a shower head EN

Intended use
The Point-of-Use sterile filter is used in medical settings to prevent infections caused by bacteria such as *Legionella pneumophila*, *Pseudomonas aeruginosa* and mycobacteria. It is used on the water faucet for the sterile filtration of tap water. The water filter protects patients receiving medical treatment, for example in oncological, transplant and intensive care wards, and also neonates, from infections caused by waterborne pathogens.

Application
The i3 TWO direct Point-of-Use sterile filter is mounted on the shower hose by means of a screw thread connection. After a 50-day maximum period of use the disposable filter must be dismantled and disposed of. Please read the operating instructions thoroughly before use.

- Safety instructions**
- Do not install the filter/accessories on in tankless (low pressure) systems
 - Do not use the filter outside of its specifications
 - The filter can be used for a maximum of 50 days
 - The duration of use may be shorter if water quality is poor (rust, sand, etc.)
 - It is not intended to be used for injection, infusion, irrigation or hemodialysis
 - The filter does not change the chemical properties nor the taste or smell of water
 - Do not remove the filter while it is in use - air in the piping system can block the filter
 - If there are any signs of contamination, then the water filter should be replaced
 - Do not put any objects (rags, towels) on the installed filter

Symbols

Article number	Latex free	Do not use if the filter or the packaging is damaged
Read the instructions thoroughly before use	Serial number	
Do not use after the expiry date	LOT number	Manufacturer

Preparation EN

Unscrew the existing shower head from the shower hose if necessary. Do not use the filter if the packaging is damaged. Do not use the filter if the use-by date has elapsed (see filter packaging).

- Installation**
1. Disinfect your hands
 2. Open the transparent vacuum packaging of the filter on the filter inlet side without touching the filter directly
 3. Insert the supplied flat gasket into the thread of the shower hose. Hold the filter in the opened package and screw it onto the shower hose
 4. Check the installed filter for leakage

- Deinstallation**
5. Unscrew the filter from the shower hose
 6. Dispose the filter with household trash or according to the local guidelines

Technical data

Bacterial retention:	> 7 log (<i>Brev. dim.</i>) per cm ² filter area
Retrograde protection:	> 4 log <i>Pseudomonas aeruginosa</i> , <i>Escherichia coli</i> , <i>Staphylococcus aureus</i> per 24 h
Flow rate:	15 l/min at 5 bar
Duration of use:	50 days (7 weeks)
Maximum operating pressure:	5 bar
Maximum operating temperature:	60 °C (max. 70 °C for 30 min)
Storage life:	5 years (at max. 40 °C storage temperature)
Chlorine resistance:	1 hour ≤ 200 ppm active chlorine; 50 days ≤ 10 ppm active chlorine
Material authorization:	Product is biocompatible as per ISO 10993-1; BS 6920:2014
Disinfection:	Can be disinfected by wiping with common surface disinfectants
Measurements:	40 x 230 mm (diameter x length)
Adaption:	G1/2" OT

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Unscrew the shower head from the shower hose if necessary. Do not use the filter if the packaging is damaged. Do not use the filter if the use-by date has elapsed (see filter packaging).

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