Your Companion
Life Science Filtration

Certification of the i3 Membrane quality management system according to DIN EN ISO 13485: 2016 and the products are EU 2017/745 (MDR) conform

i3 Membrane GmbH - Christoph-Seydel-Str. 1 - DE-01454 Radeberg
info@i3membrane.de - www.i3membrane.de
Phone: +49 (0)40-2576748-0 - Fax: +49 (0)40-2576748-48

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Information:
Duration of use: The duration of use may be shorter if water quality is poor (rust, sand etc.)
Tankless systems: Do not install the filter /accessories on tankless (low pressure) systems
Disinfection by wiping: The filter can be disinfected by wiping with the permitted disinfectant (see www.i3membrane.de/desinfection)
Disposal: The filter is discarded with household trash

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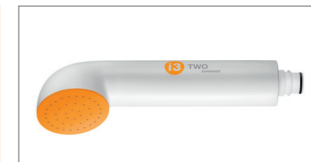
Technical data:
Bacterial retention: > 7 log per cm² (Brev. dim.)
Retrograde protection: > 4 log in 24 h (Pseudomonas aeruginosa, Escherichia coli, Staphylococcus aureus)
Flow rate: > 12 l/min at 5 bar
Duration of use: 50 days
Storage life: 5 years
Operating pressure: max 5 bar
Maximum operating temperature: 60 °C (max. 70 °C for 30 min.)
Measurements: 210 x 40 mm
Accessories: (see i3membrane.de/accessories)
Item no. 2060002 (Adapter 1/2")

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EN Instruction for use
Sterile filter for use as a shower head to prevent infection

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EN i3 TWO_{connect} sterile filter for use as a shower head

Installing

1. Make sure a quick release connector has been installed on the shower hose
2. Disinfect your hands
3. Open the transparent vacuum packaging of the filter on the filter inflow side
4. Attach the filter to the quick release connector
5. Check the installed filter for leaks

Uninstalling

1. Disconnect the filter from the quick release connector
2. Dispose of the filter with household trash

Do not use if the filter or the packaging is damaged

Read the instructions thoroughly before use

Do not use after the expiry date

Class I Medical device
Article number

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