# i3 membrane

bringing power to separation

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## i3 FIVE sterile filter for filtration of water supplied to medical devices

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### Preparation for product use

Always disinfect your hands before installing the i3 FIVE.

The i3 FIVE complies with the fundamental characteristics of a medical product and is available with a Rectus coupling on the inflow and the outflow. The sterile filter is used to filter the water supplied to medical devices such as birthing pools and cleansing and disinfecting equipment for endoscope reprocessing.

### Installation

Before installation, please check the expiry date on the packaging. Make sure that the packaging is not damaged. If the packaging is damaged or there is no visible vacuum, please use a new i3 FIVE.

Open the packaging carefully without damaging the i3 FIVE.

Insert the i3 FIVE into the Rectuscoupling in the direction of the arrow of the hose. First at the outlet side and then at the inlet side. Do not touch the connection nozzle. Check that there are no leaks. Open the water tap and ensure that water flows through the i3 FIVE. If the filter is blocked, please replace it with a new one.

The i3 FIVE is only intended for single use and can be used following installation for a maximum of 50 days. Germ retention is not guaranteed when the filter is used for longer than 50 days.

The housing of the i3 FIVE contains a bacteriostatic additive. The filter can be disinfected using surface disinfectants. The i3 FIVE can be disposed of as municipal waste.

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i3 Membrane GmbH - Bautzner Landstr. 45 - DE-01454 Radeberg