# i3 membrane

bringing power to separation

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## i3 ONE sterile filter for use at the tap

### Preparation for product use

Please disinfect your hands before installing the i3 ONE.

Please make sure that a quick-release adapter is installed at the tap.

### Installation

Before installation, please check the expiry date on the packaging. Make sure to check that the double-walled packaging is not damaged. If the packaging is damaged or there is no visible vacuum, do not use the filter and take a new i3 ONE.

Cut the double-walled packaging carefully without damaging the i3 ONE.

Insert the i3 ONE (class I medical device, sterilized) into the quick-release adapter with a slight turning movement until the filter audibly engages. For easier and safer installation, wet the O-ring with a commercial disinfectant.

Align the outlet that the water jet is not guided directly into the siphon.

Check the quick-release adapter for leak tightness.

The i3 ONE is only intended for single use and can be used following installation for a maximum of 50 days. Germ retention is not guaranteed when used for longer than 50 days.

The housing of the i3 ONE contains a bacteriostatic additive. The filter can be disinfected using surface disinfectants. The i3 ONE can be disposed of as municipal waste.