i3 membrane

bringing power to separation

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i3 TWO sterile filter for use as shower cap

Preparation for product use

Please disinfect your hands before installing the i3 TWO.

The i3 TWO (class I medical device, sterilized) is available in two versions:

1. with a quick-connection system (adapter, 1/2 inch male thread for shower hose)
2. with a screw thread.

Installation

Before installation, please check the expiry date on the packaging. Make sure that the double-walled packaging is not damaged. If the packaging is damaged or there is no visible vacuum, do not use the filter and take a new i3 TWO.

Cut the double-walled packaging carefully without damaging the i3 TWO.

1. Using the i3 TWO with quick-connection system:

Insert the sterile filter into the adapter of the shower hose.

2. Using the i3 TWO with screw thread:

Screw the sterile filter directly onto the hose thread.

Check the adapter or hose thread for tightness. Open the water tap and ensure that the i3 TWO operates reliably. If the filter is blocked, please change the sterile filter.

The i3 TWO is only intended for single use and can only be used following installation for a maximum of 50 days. Germ retention is not guaranteed when used for longer than 50 days.

The housing of the i3 TWO contains a bacteriostatic additive. The filter can be disinfected using surface disinfectants. The i3 TWO can be disposed of as municipal waste.